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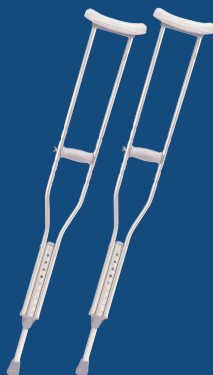
Patient Height  
Altura del paciente  
Hauteur du patient  
4'0" to 4'6"

### CAUTIONS:

- Use after proper training under the direction of your Doctor or Physical Therapist.
- Replace Crutch Tips if worn.
- Keep Hand Grip Wing Nut tightened.
- Use extra caution on wet and slippery surfaces.
- Keep hand grip wing nut tightened.

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EU Authorized Representative  
Drive International, LLC  
Globe Lane - Stockport  
DUKINFIELD SK16 4RE, GB

# aluminum crutches pediatric

Push-Button/Adjustable Fully Assembled

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**For maximum fit and assembly, see instructions below:**

1. **Height:** Adjust height by pressing both push-buttons, rotating foot-piece, and pulling down until desired patient height is obtained. To help in doing this, the foot-piece has a circular ring marked to indicate location of push-button. Rotate foot-piece again until both push buttons snap back into place.
2. **Handgrip:** Adjust handgrip to fit an approximate 10" bend at patient elbow.
3. **Safety Check:** With crutches positioned per diagram below, confirm patient can lean forward comfortably while resting on the underarm supports. Patient should not have to stand uncomfortably erect. Ensure both push-buttons are firmly in place and handgrip nuts are tightened securely.

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