


Sliding SHOWER

- ▶ Can be used as a stationary or handheld shower
- ▶ Sliding bar offers comfort and convenience at various heights
- ▶ Three spray settings: full body, full body with pulsating massage and pulsating massage
- ▶ Clear plastic sliding soap dish
- ▶ Flexible 60" chrome hose
- ▶ Fits standard 1/2" threaded shower arm

1. Locate position on your wall where bar will be placed. Using the template provided mark screw holes, pre-drill and insert anchors flush with wall surface. Bar should be placed so hose can reach water source.

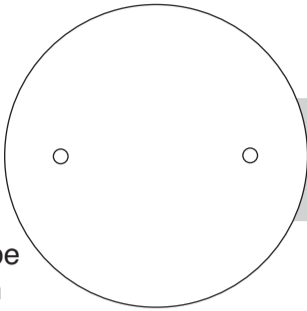

**Top
SCREW
POSITIONING
Template***

*** PLACE THIS CARD ON SHOWER WALL TO MARK SCREW HOLES FOR SHOWER BAR WITH TEMPLATE.**

2. Remove chrome end caps from both bar supports and put aside. Remove white plastic end caps. Using predrilled holes with anchors, insert small screws through white plastic end caps and secure to wall surface. Place bar support over white end caps and insert large screw. Insert bar and replace chrome end cap.

3. Slide shower mount onto shower mount bar. (Optional plastic soap dish can also be slid onto the bar first so dish is at the bottom). Place bar into chrome supports and place supports back over plastic end caps which have been fastened to your shower wall.

4. Take the longest screw and place through bar support and screw into plastic end caps. Replace chrome end caps back onto chrome bar supports.

5. Screw the large end of the water hose onto shower head and the small end to your water outlet pipe.

6. Place shower head into mount and tighten for stationary use. Shower can also be used as a handheld.

7. Shower mount will slide up and down the bar to the desired height needed. Loosen shower mount to slide, tighten to secure in place.


Distributed by:
 Duro-Med Industries
 Div. of DMS Holdings Inc.
 1931 Norman Drive South
 Waukegan, IL 60085 USA
 www.mabisdmi.com
 Made in Taiwan R.O.C.
 #921-1584-0600 11/08
 ©2008 Duro-Med Industries



DMI believes in 100% customer satisfaction
 Customer Care Help Line: 877-505-6190
 Monday - Friday: 8:00 am - 4:30 pm CST

523-1584-0600

0 4 1298 01584 0

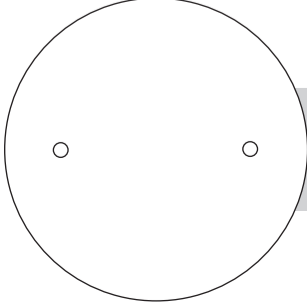

**Bottom
SCREW
POSITIONING
Template***