INNOVATIVE DESIGNS
for independent living™

DN8001 PAUSE CONTROL HANDHELD SHOWER INSTALLATION INSTRUCTIONS DUCHA DE MANO CON CONTROL PARA PAUSA INSTRUCCIONES DE INSTALACIÓN DOUCHE À MAIN AVEC BOUTON DE PAUSE DIRECTIVES D'INSTALLATION

The Pause Control handheld Shower comes packed with a free Handheld Shower Attachment for use with Home Care by Moen Adjustable Chair, Part number DN7060 or Transfer bench, part number DN7065.

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Remove the existing cradle or showerhead from the wall by unscrewing (counter clockwise) the product from the end of the shower arm.

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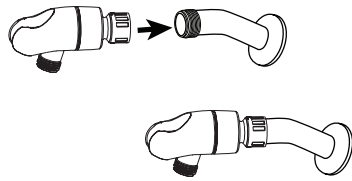
Figure 1
Figura 1
Illustration 1

1. Cover the thread of the pipe coming out of the wall with Teflon® tape (included). Screw the Pause Control Handheld shower cradle on to the pipe (shower arm) coming out of the wall.

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2. Identify the end of the hose with the black washer inside the plastic nut and screw the hose onto the bottom of the cradle (see figure 2).

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Figure 3
Figura 3
Illustration 3

Figure 4
Figura 4
Illustration 4

3. Screw handheld shower onto the other end of the hose (figure 3).

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4. Rest the handheld shower in the cradle (figure 4).

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WARNING: Follow these installation instructions carefully. Proper installation is the installer's responsibility. Failure to follow correct installation procedures may cause the handheld shower to become loose, which could result in serious injury. Contact CSI immediately at 1-800-882-0116 if any components are missing or damaged. Assemble in accordance with CSI's instructions. Do not make substitutions or modify this product in any way. FOR QUESTIONS CONTACT CSI AT 1-800-882-0116.

LIMITED LIFETIME WARRANTY: If this product fails due to a defect in materials or workmanship at any time during the life of the product, Creative Specialties International, a division of Moen Incorporated (CSI) will replace it free of charge, postage-paid. Simply contact CSI at 1-800-882-0116 for replacement information. This warranty does not cover products which have been abused, altered, damaged, misused, cut or worn. TO THE EXTENT PERMITTED BY LAW, CSI DISCLAIMS ALL OTHER IMPLIED OR EXPRESS WARRANTIES INCLUDING ALL WARRANTIES OF MERCHANTABILITY AND/OR FITNESS FOR A PARTICULAR PURPOSE.

LIMITATION OF LIABILITY: THIS WARRANTY IS YOUR SOLE REMEDY AND CSI SHALL NOT BE LIABLE FOR ANY DAMAGES, WHETHER DIRECT, INDIRECT, INCIDENTAL, SPECIAL, CONSEQUENTIAL, EXEMPLARY, OR OTHERWISE, INCLUDING LOST REVENUES AND LOST PROFITS, ARISING OUT OF ANY THEORY OR RECOVERY, INCLUDING STATUTORY, CONTRACT OR TORT. NOTWITHSTANDING THE TERM OF ANY LIMITED OR IMPLIED WARRANTY, OR IN THE EVENT THAT ANY LIMITED WARRANTY FAILS OF ITS ESSENTIAL PURPOSE, IN NO EVENT WILL CSI'S ENTIRE LIABILITY EXCEED THE PURCHASE PRICE OF THIS PRODUCT. SOME STATES AND PROVINCES DO NOT ALLOW THE EXCLUSION OR LIMITATION OF INCIDENTAL OR CONSEQUENTIAL DAMAGES SO THE ABOVE LIMITATIONS OR EXCLUSIONS MAY NOT APPLY TO YOU. THIS LIMITED WARRANTY GIVES YOU SPECIFIC LEGAL RIGHTS, AND YOU MAY ALSO HAVE OTHER RIGHTS WHICH VARY FROM STATE TO STATE AND PROVINCE TO PROVINCE.

For additional warranty information, visit www.homecare.moen.com.

The Home Care Pause Control Handheld Shower meets or exceeds the applicable requirements of the following standards / guidelines:

 The Americans with Disabilities Act (ADA)

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Creative Specialties is a division of Moen Incorporated. © 2005
Home Care is a registered trademark of Home Care Tim-BR-MART, licensed to Moen Incorporated.
Home Care and Creative Specialties are registered trademarks of Moen Incorporated.