

# I N S T R U C T I O N S



## Hinged Elevated Toilet Seat

Catalog Number 72571 Series


U.S. Patent #5,251,338

Other Patents and Int'l. Patents Pending

1. Remove existing seat and lid from the toilet.
2. Place gasket (C) on toilet with holes in the toilet aligned with the holes in the gasket.
3. Place Hinged Toilet Seat Unit (A & B) on toilet, small slotted holes up, making sure the holes line up with the holes in the gasket.

4. Place existing toilet seat over the hinged toilet seat.
5. Insert the bolt (D) through the toilet seat hinge, the mounting block of the hinged seat (B), the gasket (C) and the toilet.

6. Repeat with the second bolt (D) in the other hole.
7. Slip washer (E) onto the end of the bolt (D) and tighten nut (F). Repeat with second bolt.

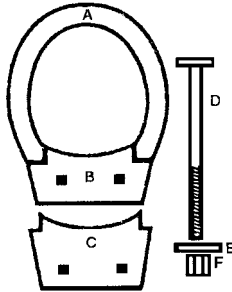

- Parts
- A. Lifting Riser
  - B. Mounting Block
  - C. Gasket
  - D. Seat Bolt
  - E. Washer
  - F. Nut



### WARNING:

**HINGED TOILET SEAT SHOULD NEVER BE USED WITHOUT THE NORMAL TOILET SEAT ATTACHED AND HINGED IN THE DOWN POSITION**

# Maddak

6 INDUSTRIAL ROAD • PEQUANNOCK, NJ 07440-1993 USA

973-628-7600 • fax 973-305-0841 • email [custservice@maddak.com](mailto:custservice@maddak.com) • [www.maddak.com](http://www.maddak.com)